

# ANSWER<sup>2</sup>

## Children & Adolescent Sizing Chart

**Important:** The ANSWER 2 shoes arrive with removable inserts that increase your fitting capabilities, be certain to remove the inserts before attempting to fit the shoe to the brace side.

To check that your sizing chart printed accurately, the big rectangle box should measure 4" wide x 10" high.

### Length Measurement

#### 1 Place heel on chart

- Align heel of foot/AFO/KAFO with the bottom line.
- If foot is longer than brace, include foot


#### 2 Trace foot/AFO/KAFO

- Match longest point to number on right
- If you are between or slightly above line, round up

#### 3 Necessary Adjustments for Circumference

For all AFOs, DAFO<sup>®</sup>s, KAFOs, SMOs with:

- If you are between or slightly above line, round up:  
**Increase 1/2 size**
- Articulating hinges, posterior stop and instep strapping:  
**Increase 1 full size**
- Toe strapping, the additions above will change the fit of the shoe:  
**Increase 1/2 size**

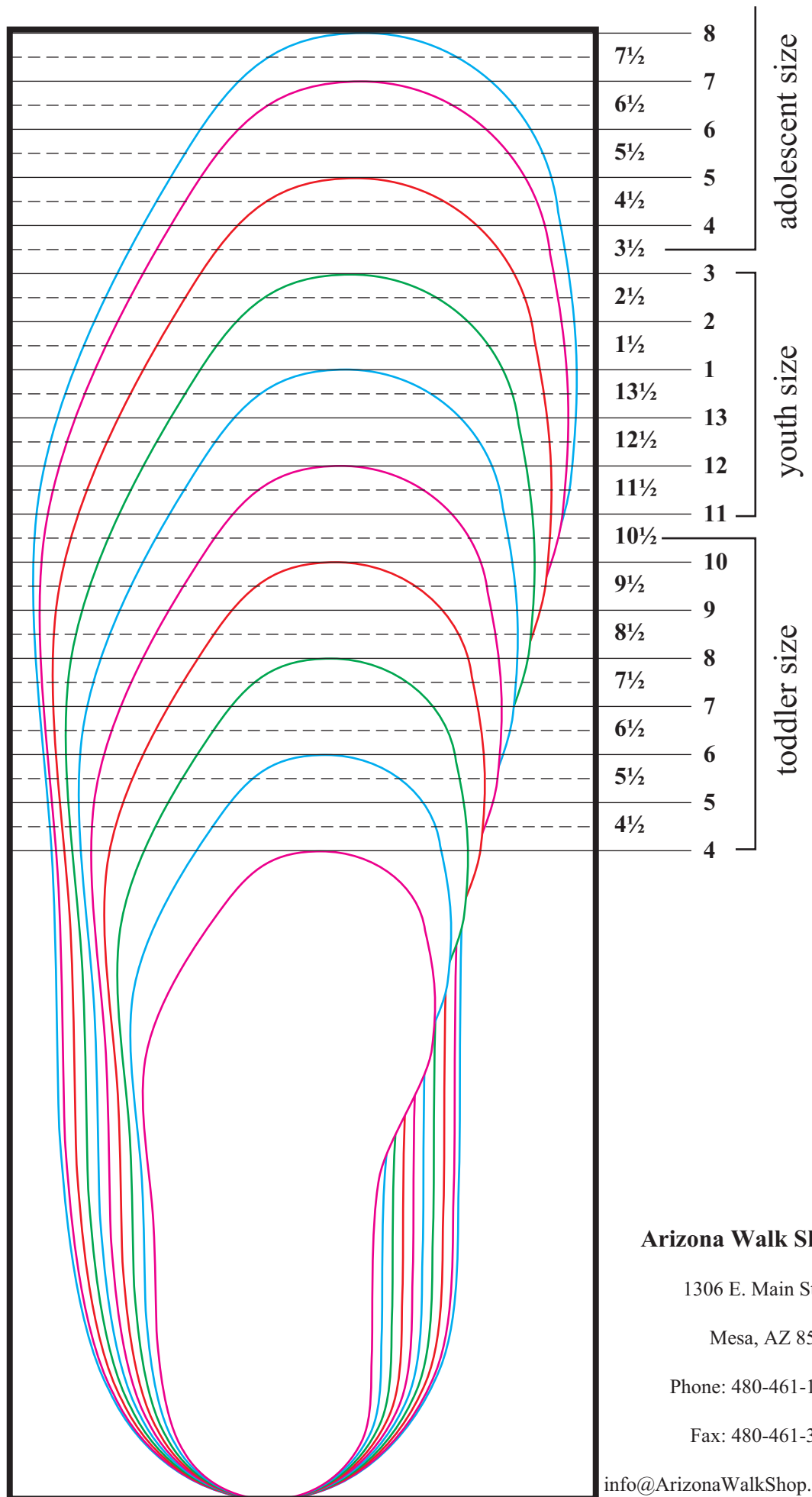

### Arizona Walk Shop

1306 E. Main Street

Mesa, AZ 85203

Phone: 480-461-1940

Fax: 480-461-3855

info@ArizonaWalkShop.com

Place Heel Here